

Invacare® | **Softform® Premier
Active Cushion**



Static Dynamic Cushion

The *Invacare Softform Premier Active Cushion* has been designed for patients who are considered at 'High Risk' of developing pressure ulcers and require pressure reduction 24 hours a day. The unique design features an alternating air insert beneath the castellated foam of the **Softform Premier Cushion**, transforming the static cushion to a dynamic surface in seconds and offering the patient two cushions in one. For convenience, the alternating cushion operates from the same pump as the **Softform Premier Active Mattress** ensuring the user has continuity of care.



Yes, you can.®

An innovative static dynamic cushion

The air insert contained within the *Invacare Softform Premier Active Cushion* allows stepping up and down between static and dynamic alternating systems with minimum patient disruption.

The air cells work on a two up, two down function beneath the insert, delivering a very low impact at the point of change, thereby delivering high levels of comfort in addition to improved pressure reduction.



How it works:



To step up a patient, retract the pipes from the cushion.



Fit pipes to the Active pump. Turn pump on – no additional adjustments required.



Patient is now on an alternating surface.

Features and Benefits



Compatible with Active Mattress
The same pump can be used for both, offering a 24 hr support system.



Pressure can be maintained at a level appropriate to individual patients. Fully alarmed in case of power failure.

INVACARE® Limited
Pencoed Technology Park - Pencoed - Bridgend CF35 5HZ
Tel: +44 1 656 776 222
Fax: +44 1656 776 220
E-mail: uk@invacare.com
www.invacare.co.uk
Sales Order E-mail: ordersuk@invacare.com

INVACARE® Ireland Ltd
Unit 5 - Seatown Business Campus
Seatown Road - SWORDS - County Dublin
Tel. +353 1 8107084
Fax +353 1 8107085
Email: Ireland@invacare.com
www.invacare.ie

© 2011 Invacare® International Sarl
All rights reserved.

All information quoted is believed to be correct at time of print. Invacare® reserves the right to alter product specifications without prior consultation.

Softform Premier Active Cushion
UK - 01/2011 - 1524062




Yes, you can.®


Technical data



Softform Premier Active Cushion 460 mm

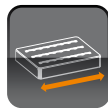

510 mm

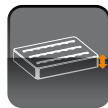

152 mm



1.5 kg



Max. User Weight:
17 stone (108 kg)



Softform Premier Active Cushion Cleaning: 80°

Product codes:

Standard Active Cushion	1523384
Air Insert	1523383
Active Pump	1486172

Warranty:

Cover:	12 months
Foam:	12 months
Pump:	24 months
Air Insert:	12 months

Manufactured to European Standards:
Foam: BS 5852: Crib 5; Cover BS7175: Crib 5.



Fuchsia (PU)